



ENDODONTICS



01274 885544
www.trycare.co.uk/tehnodent

Trycare

Edetale Gel

With Peroxide

Using a file lubricant reduces stress on the instrument by softening the dentine. Simply coat the instrument after each cleaning and withdrawal cycle. Edetale Gel softens the dentine making shaping easier, the peroxide's effervescent action pushes detritus coronally. Used alternately with Antiseptic Liquid No.2 kills bacteria and removes the smear layer.



5ml syringe **£13.95**
Product Code **UTEHN02**

Also available: Edetale Gel without Peroxide Syringe 5ml £9.50

Product Code **UTEHN03**

Edetale EDTA Liquid

20% EDTA with Cetrimide

Sclerotic canals such as those found in elderly patients can be effectively enlarged by applying Edetale liquid directly into the canal system. The material can be left in situ to allow it to chelate with Ca⁺ from the apatite thereby softening it. Edetale Liquid can also be used for the detection of root canal entrances where it will soften the most recently calcified dentine. It contains powerful surfactants and surface acting antiseptics which enable fast penetration of the medication into the root canal and dentinal tubules. In the case of difficult sclerotic canals it can be left on a cotton wool pellet or paper point, to find and open accessory canals.



100ml bottle **£17.95**
Product Code **UTEHN04**

Antiseptic Liquid No.2

3% Sodium Hypochlorite

Hypochlorite 3% Solution is used to irrigate root canals during debridement and shaping. Antiseptic Liquid No. 2 removes the "smear" layer and penetrates the root canal system disinfecting it. Upon reaching the apex copious irrigation of the periapical area will disinfect the canal.

Please note:

Hypochlorite solution once opened oxidises on exposure to air, so close the bottle immediately after use. For this reason, to ensure a solution with maximum activity it is better to buy smaller bottles, it is also available in a 5.2% strength which may suit some users.



100ml bottle **£7.50**
Product Code **UTEHN08**

Cresotin Liquid No.2

With Pipette dispenser

A well tried formulation which enables rapid mummification of teeth with acute pulpitis. Use of Lignocaine anaesthetic, in the first instance, is suggested, to alleviate acute pain. Applying the solution to a "hot pulp" Cresotin Liquid No.2 will "fix" the pulp tissues making them friable, parchment like and easily removed. The root canal system will be effectively rendered sterile.



15ml bottle **£19.95**
Product Code **UTEHN05**

Partial substitute for the no longer available Cresophene, does not contain, the steroid, Dexamethasone

Resortin

Treatment of narrow difficult infected root canals.

Where the narrowness of the canal is such, that there still maybe remains of the pulp. Mix 1 drop of each liquid then add 2 doses of powder. Introduce into the difficult canal Resortin has a working time of 20 mins and will fully harden in 4/5hrs. Powerful anti-bacterial and mummification properties are locked in when fully hardened.



Contains: 1 x 5ml Therapeutic
1 x Catalyst and 1 x 10g Powder
Product Code **UTEHN21**

Calcetin Calcium Hydroxide

Cavity Liner Paste

Wherever the stimulation of the production of secondary dentine needs to be induced. Calcetin produces a high pH 12.8 and make Ca+ ions available for the laying down of secondary dentine. Use in all cases of pulp exposure to cover the exposure. For apexification, leave in place within the canal. Calcetin is radio opaque and is easily removed from canals if necessary. Ultra thin dispensing tips 0.52 diameter to enable direct injection of paste into the canal.



2gm syringe + tips **£14.95**
Product Code **UTEHN10**

Dehydrating & Degreasing Liquid

The final rinse in Root Canal Therapy.

A hygroscopic formulation with low surface tension allows deep penetration of microtubules drying and removing water. It should be used as the final rinse after copious irrigation with Hypochlorite solutions such as Antiseptic Liquid No.2. Simply apply, then dry using paper points.



15ml Bottle
Product Code

£12.95
UTEHN09

Rootdent

Enables the repair of transposed, zipped and fractured root canals.

Rootdent is generic MTA in the same formulation. A complex of the oxides of Calcium, Silicon and Aluminium. Rootdent makes a previously expensive material, used by Endodontists, for retreatments more affordable for GDP's.

It is extremely important to thoroughly disinfect the area to which the Rootdent is to be applied. As acid producing bacteria will inhibit its action.

The original MTA patents HAVE EXPIRED which means you no longer have to pay £200+



10 x 0.5gm
Product Code

£95.00
UTEHN01

Eugetin Root Canal Sealer

Formulated with Zinc Oxide, Calcium Phosphate and Di-iodothymol (antimicrobial). Unlike the old standard Sargenti and Resin type pastes it does not contain or produce formaldehyde.

It does however contain very low doses of two steroids, dexamethasone 0.01% and hydrocortisone 1.00% for relief of the inflammatory response to root canal therapy.

Can be used with GP points or as a simple paste. A film thickness of 3-5 microns allows good penetration into dentinal tubules.



14gm Jar + 10ml Bottle with Pipette cap £35.95

Product Code

UTEHN11

Hemostop

Achieves clotting in 10 seconds!

A 25% solution of Aluminium Chloride with Cetrimide. Use with applicator stick or retraction cord or wherever you need to stop blood flow. Aluminium Chloride acts to precipitate blood proteins in the cut capillaries without producing distal necrosis.



Also available: Hemostop Gel 5ml **£13.95**
Product Code **UTEHN07**

15ml Bottle with Pipette **£17.95**
Product Code **UTEHN06**

Desensetin

Immediate relief of sensitivity

Apply directly to the sensitive area. The unique combination penetrates micro-defects in enamel and deep into the dentine. Interaction of the components with saliva remineralises defects and occludes dentinal tubules. Thereby reducing sensitivity.

Desensetin is safe for patients to purchase and apply at home.



5ml Syringe **£9.95**
Product Code **UTEHN12**

Ftor-Lux Fluoride Varnish

A two bottle system- 10ml bottles

Cervical hypersensitivity, reduce the sensitivity of vital teeth prepared for a crown or other restorative procedure. After deep scaling, prophylaxis and tooth whitening.

Benefits: Immediate pain reduction for patients. Promoting hard tooth tissues remineralisation. High concentration of fluoride ions.



2 x 10ml Bottles **£29.95**
Product Code **UTEHN13**

Poliren Polishing Paste No. 1

Quick softening and removal of tartar.

For heavy calculus. Protect gingiva with vaseline apply in place for 3-5 mins then scale. Mild acid softens plaque and calculus. Scale and polish. Apply Ftor-Lux Fluoride Varnish to remineralise surface.



45gm Jar
Product Code

£15.95
UTEHN14

Poliren Polishing Paste No. 2

Effective professional enamel cleaning.

Heavy duty, prophylaxis paste contains carborundum and quartz. Use before tooth whitening procedures and other cosmetic work.



45gm Jar
Product Code

£15.95
UTEHN15

Buy both at £27.95 + VAT

Trycare

www.trycare.co.uk