



UNIVERSAL-XL
200 EXTRA LARGE WIPES
267 x 210mm

V-WIPE ZERO

Surface Disinfectant Wipe

ALCOHOL FREE



2in1
DISINFECTS & CLEANS

STAY FRESH LID

CONTACT 60 SEC

APPLE AROMA

KEY FEATURES

Introducing the new surface disinfectant wipe flowpack from Bossklein, combining the dispenser tub and refill pack into a single, economical packaging platform.

Each pack includes an easy open and sturdy stay fresh lid. This ensures the next wipe remains moist and ready for the next application.

All Bossklein V-Wipe Zero Surface Disinfectant Wipes are dual registered. They are registered for use on non-invasive medical devices and they are also a Biocide registered product, enabling these wipes to be used in all types of clinical and non-clinical environments.

The brand new, alcohol free Zero formulation is ideal to use on the majority of wipeable surfaces even those that are sensitive to alcohol, including dental chairs*

WHAT YOU NEED TO KNOW

- Alcohol free surface disinfectant wipes
- Contact time of only 60 seconds
- Dual registration; Medical Device & Biocide
- Effective against Coronavirus
- Wide effective spectrum
- Safe to use on sensitive surfaces*
- 2 in 1; Cleans and disinfects
- Subtle apple aroma

The vigorously tested solution is effective against multiple micro-organisms, including all Bacteria and Yeast. EN 14476 Virucidal testing proves efficacy against all enveloped viruses including Coronavirus, HIV, Hepatitis and Influenza in just 60 seconds.

V-WIPE ZERO is ideal to use where time is critical, such as in between patients and also in high traffic areas.

COMPOSITION

Contains Aqua, Alkyl (C12-16) dimethyl/benzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecyldimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes. Liquid weight 1050g.

INSTRUCTIONS FOR USE

Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action over 60 seconds. Allow surface moisture to dry or wipe with clean tissue. For professional use only.

* Always test an inconspicuous area first. Do not use on copper or aluminium.



Complete hygiene solutions

DirectaDentalGroup
DIRECTA topdental ORSING parkell physics
RÖNIG TrolDental Sendoline CONTACTEZ

Topdental (Products) Ltd
12 Ryefield Way Silsden West Yorkshire BD20 0EF UK
Tel +44 (0)1535 652 750 www.topdental.co.uk

V-WIPE ZERO

Surface Disinfectant Wipe

- Alcohol free formulation
- Pack of 200 wipes
- Extra large wipe 267 x 210mm
- Contact time of only 60 seconds
- Highly effective disinfectant
- Safe to use on most surfaces*



A SAFER & CLEANER APPROACH

The revised formula takes advantage of modern, safer and greener chemical components to create a more simple and effective product for the end user to utilise in the ongoing daily fight against infection.

The new ZERO formulation contains a synergistic blend of two powerful biocides, is free from PHMB and contains detergent derived from renewable resources.



EXTRA STRONG

WIPE SUBSTRATE

The largest wipe available in the Bossklein V-WIPE range, each Universal-XL wipe measures 267 x 210mm. Each pack contains 200 extra large wipes.

The textured surface of the 34gsm non woven polypropylene wipe material is extra durable and perfect for clinical disinfection. By transferring as much disinfectant solution to the non porous surface of choice as possible, this strong, low-linting material is the ideal infection control delivery method for surfaces.



UNIVERSAL-XL
200 EXTRA LARGE WIPES
267 x 210mm

MICROORGANISM	CONTACT TIME
Bactericidal	60 sec
BS EN 16615:2015	
BS EN 13727:2012 + A2:2015	
Pseudomonas aeruginosa	60 sec
Staphylococcus aureus	60 sec
Enterococcus hirae	60 sec
Yeasticidal	60 sec
BS EN 16615:2015	
BS EN 13624:2013	
Candida albicans	60 sec
Enveloped viruses	60 sec
BS EN 14476:2013 + A2:2019	
Vaccinia virus	60 sec
Hepatitis B virus (HBV)	60 sec
Hepatitis C virus (HCV)	60 sec
Hepatitis Delta virus (HDV)	60 sec
HIV	60 sec
Influenza virus	60 sec
Coronavirus	60 sec
Herpesviridae	60 sec
Filoviridae (Ebola)	60 sec
Rubella virus	60 sec
Rabies virus	60 sec
Measles virus	60 sec
Poxviridae	60 sec



BACTERICIDAL



YEASTICIDAL



VIRUCIDAL*

EN14476
VIRUS TESTED

MICROBIOLOGICAL EFFICACY

Bactericidal, Yeasticidal and Virucidal against all enveloped viruses including Coronavirus, Hepatitis B and C and HIV. This highly effective formula has been tested to a variety of standards including EN 14476 for proven efficacy.

The new Bossklein ZERO surface disinfectant is effective against all listed (opposite) micro-organisms in only 60 seconds. This ensures complete peace of mind when it comes to clinical surface disinfection.

The following test standards have been achieved: EN 16615, EN 14885, EN 14476, EN 13727, EN 13624.

VARIANTS AND PACKAGING

BOS4001 Flowpack of 200 Universal-XL Wipes x 6 per box



Complete hygiene solutions

DirectaDentalGroup
DIRECTA topdental ORSING parkell physioCS
RØNIG TrolDental Sendoline CONTACTEZ

Topdental (Products) Ltd
12 Ryefield Way Silsden West Yorkshire BD20 0EF UK
Tel +44 (0)1535 652 750 www.topdental.co.uk