

Lisa[®] deminir₂₀



Demineralizer



Purified water for optimal results

The quality of the water used in a sterilizer is of the greatest importance.

Good quality demineralized water guarantees consistent, optimal performance and extends the life of your sterilizer and of your instruments.

Lisa demin 20 directly connects to your tap water supply and produces demineralized water of exceptional quality.

Automatic water supply

Lisa demin 20 connects directly to the LISA sterilizers, allowing for the automatic filling and drainage of used water. This direct connection eliminates risks of error by the final user and saves time.

No monitoring or maintenance

Lisa demin 20 doesn't need any monitoring or maintenance. Water quality is checked during supply, and a warning is displayed on the touch screen of the Lisa sterilizer when the demineralizing resin cartridge needs to be replaced.

Safety

Lisa demin 20 is delivered with the **Water block®** safety system, which interrupts the water supply in the event of excess flow through pipe breakage. It is therefore not necessary to open and close the tap daily.


Quantity of water produced

The quantity of water produced per resin cartridge depends upon the quality and hardness of the tap water.

On average it varies from 80 to 140 litres, which is equivalent to 240 to 420 sterilization cycles per cartridge (Lisa MB17).

Compatibility chart

Lisa demin 20 is compatible with our previous sterilizer models by using the appropriate retrofit kit(s).

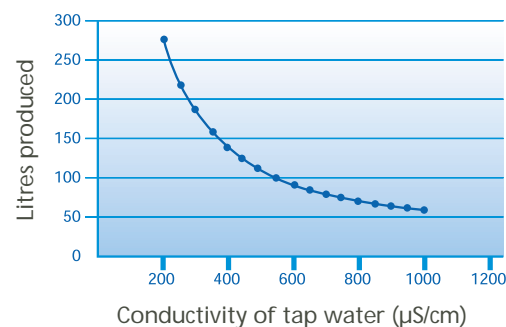


Model	Serial Number	Retrofit kit(s)
LISA MB 17	< 01-2107	Kit 1
	from 01-2107 to 03-1462	Kit 2 + Kit 3
	from 03-1463 to 03-2789	Kit 3
	> 03-2789	No kit necessary
LISA MB 22	from 02-0001 to 03-0373	Kit 2 + Kit 4
	from 03-0374 to 04-0125	Kit 2
	> 04-0125	No kit necessary

Accessories

Resin cartridges	A8120000 (2 pieces)
Retrofit Kits	Kit 1 : X0500240 / Kit 2 : X0500210 Kit 3 : X0500200 / Kit 4 : X0500230

Technical Specifications

Min. - max. tap water pressure	1 - 5 bar
Dimensions	L : 680 mm / W : 190 mm / D : 220 mm
Weight (total/cartridge)	3.6 Kg / 1.6 Kg
Power supply	24 V DC supplied by the sterilizer
Maximum power consumption	9 W (Insulation Class III)

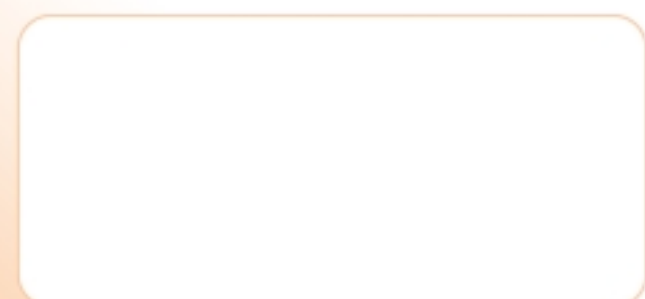

STERILIZATION

W&H Sterilization S.r.l.
Italy I-24066 Pedrengo (BG)
Via delle Crocette, 2
☎ +39 / 035 / 66 94 111
☎ +39 / 035 / 66 83 87
✉ office@wnhster.it
🌐 www.wh.com

