

# Pentamix™ Lite

Automatic Mixing Unit



## Affordable and portable mixer.

### Benefits:

- **Homogeneous and void-free mixing results** of reproducible quality.
- **Fast, time-saving** procedure and push-button activation.
- **Economical** and exact dispensing of the amount of material needed.
- **Hygienic** direct filling of tray and syringe without the risk of cross-contamination.
- **More efficient workflow** for higher productivity.

### Pentamix™ Lite Automatic Mixing Unit – the entry model to the world of automatic mixing.

With its unique swivelling design, the Pentamix™ Lite Automatic Mixing Unit requires a minimum of space in the upright standby position and can be stowed in a space-saving way. To swivel it back to the working position, one easy movement will do. Thanks to its low weight and its ergonomic handgrip, the device can also easily be carried from one operator to another – if desired. Simplify your life: start automixing!



#### COMPACT design

Saves valuable space in the upright standby position.



#### EASY to get started

Fast and intuitive set-up – just swivel in the working position.



#### CONVENIENT & CLEAN operation


Press the green start button and start filling the tray or syringe directly from the device.




#### PORTABLE design

Easy to carry from operator to operator thanks to light weight and integrated handgrip.


## Fast and intuitive set-up AND USE

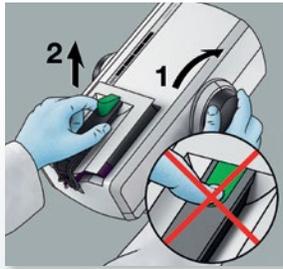

1. Remove the cartridge.

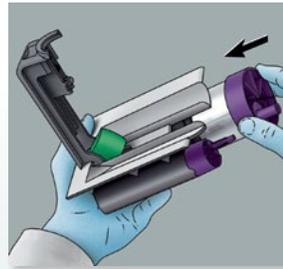


2. Insert Penta™ Foil Bag with the 3M ESPE impression material of your choice into the cartridge.

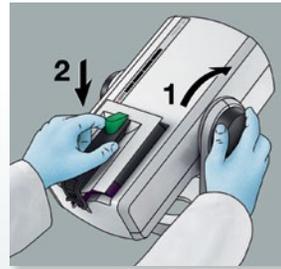

3. Insert the cartridge into the Pentamix™ Lite Automatic Mixing Unit.




4. Turn the wheel (piston) forward.

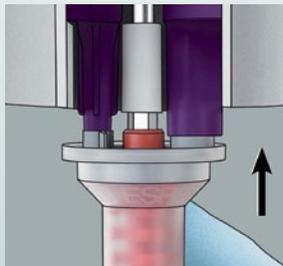

5. Attach a Penta™ Mixing Tip – Red.



6. Close the latch with an audible click.

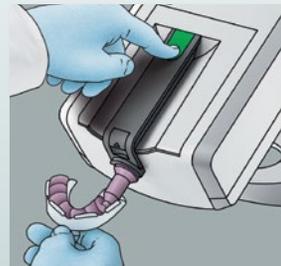

7. Press the green start button to fill the impression tray or elastomer syringe.

## Ordering Information

### PENTAMIX™ LITE AUTOMATIC MIXING UNIT AND ACCESSORIES

#### Item No. Product Information

77907	<b>Pentamix™ Lite Automatic Mixing Unit – 120 V/60 Hz</b> Includes: 1 Penta™ Cartridge for Pentamix™ Lite
77944	<b>1 Penta™ Cartridge for Pentamix™ Lite</b>
77949	<b>Penta™ Mixing Tips – Red</b> Includes: 50 Mixing Tips

### IMPREGUM™ SOFT POLYETHER IMPRESSION MATERIAL

Refills: Each contains: 2 – 300ml base pastes, 2 – 60ml catalysts

#### Item No. Product Information

31731	<b>Impregum™ Penta™ Soft Medium Body</b>
31741	<b>Impregum™ Penta™ Soft Heavy Body</b>
31765	<b>Impregum™ Penta™ Soft Quick Step Medium Body</b>
31769	<b>Impregum™ Penta™ Soft Quick Step Heavy Body</b>
31746	<b>Impregum™ Penta™ Soft Light Body</b> Contains: 1 – 300ml base pastes, 1 – 60ml catalysts
31785	<b>Impregum™ Soft Tray Material</b> Contains: 4 – 50ml cartridges, 10 – polyether tray mixing tips, purple

### IMPRINT™ 4 VPS IMPRESSION MATERIAL

Penta™ Refills — Tray Material: Each contains: 2 – 300ml base pastes, 2 – 60ml catalysts

#### Item No. Product Information

71484	<b>Imprint™ 4 Penta™ Heavy</b>
71485	<b>Imprint™ 4 Penta™ Super Quick Heavy</b>
71486	<b>Imprint™ 4 Penta™ Putty</b>

Refills — Wash & Tray Materials – Cartridge: Each contains: 4 – 50ml cartridges, 5 – yellow mixing tips, 5 – green 3M™ ESPE™ Intra-Oral Syringes

#### Item No. Product Information

71487	<b>Imprint™ 4 Regular</b>
71488	<b>Imprint™ 4 Light</b>
71489	<b>Imprint™ 4 Super Quick Regular</b>
71490	<b>Imprint™ 4 Super Quick Light</b>

### POSITION™ PENTA™ QUICK VPS IMPRESSION MATERIAL

Refill: Contains: 2 – 300ml base pastes, 2 – 60ml catalysts

#### Item No. Product Information

29063	<b>Position™ Penta™ Quick VPS Alginate Replacement – Refill</b>
-------	---

Customer Care Center: 1-800-634-2249

[www.3MESPE.com/PentamixLite](http://www.3MESPE.com/PentamixLite)



Review these products at [www.3MESPE.com/review](http://www.3MESPE.com/review)

**3M ESPE**

#### Dental Products

2510 Conway Avenue  
St. Paul, MN 55144-1000 USA  
1-800-634-2249

#### 3M Canada

Post Office Box 5757  
London, Ontario N6A 4T1  
Canada  
1-888-363-3685

3M, ESPE, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH.  
Used under license in Canada. © 3M 2014. All rights reserved.  
Please recycle. Printed in USA.  
70-2013-0603-5