Sof-Lex™

Finishing and Polishing System Procedure Guide

Sof-Lex System Procedure

The Sof-Lex System is color-coded from dark (coarse) to light (superfine) for an easily followed step-by-step process. Easy, pop-on mounting lets you quickly change discs or spiral wheels—so you can move efficiently through your finishing and poliching procedure.

and	polishing procedure.							
	policinity procedure.	Sof-Lex™ Discs	Sof-Lex™ XT Discs	Sof-Lex™ Strips	Sof-Lex™ Spiral Wheels	Anterior	Posterior	Interproximal
Gross Reduction	 Excess removal Shape anatomy Remove air-inhibited layer	•				Sof-Lex™ Discs, Coarse Rotating coarse or medium contouring instruments (e.g., diamonds, carbide burs, stones)	Rotating coarse or medium contouring instruments (e.g., diamonds, carbide burs, stones)	Sof-Lex™ XT Discs, Coarse Sof-Lex™ Strips • Coarse/Medium • Coarse/Medium, Narrow
Final Contour	Reduce surface roughness Complete and refine contours and margins	•				Sof-Lex™ Discs, Medium Rotating medium or fine contouring instruments (e.g., yellow or red ring diamonds)	Rotating medium or fine contouring instruments (e.g., yellow or red ring diamonds)	Sof-Lex™ XT Discs, Medium Sof-Lex™ Strips • Coarse/Medium • Coarse/Medium, Narrow
Finish	Remove scratchesSmooth surfacePre-polish	0				Sof-Lex™ Spiral Finishing Wheel Sof-Lex™ Discs, Fine	Sof-Lex™ Spiral Finishing Wheel	Sof-Lex™ Spiral Finishing Wheel Sof-Lex™ XT Discs, Fine Sof-Lex™ Strips, Fine/Superfine
Polish	• Final polish to high gloss	0				Sof-Lex [™] Spiral Polishing Wheel Sof-Lex [™] Discs, Superfine	Sof-Lex™ Spiral Polishing Wheel	Sof-Lex [™] Spiral Polishing Wheel Sof-Lex [™] XT Discs, Superfine Sof-Lex [™] Strips, Fine/Superfine





Review these products at www.3MESPE.com/review