Bossklein

www.bossklein.com





SDactiv^{ZERO}

Surface Disinfectant

ALCOHOL FREE







KEY FEATURES

Bossklein SDactiv ZERO Surface Disinfectant is a highly effective cleaner and disinfectant spray. Designed primarily for the dental and medical sector.

The brand new alcohol free ZERO formulation is ideal to use on the majority of non porous, wipeable surfaces; even those that are sensitive to alcohol, including dental chairs*

- 2 in 1; Cleans and disinfects
- Safe to use on sensitive surfaces'
- Modern, safe formulation
- Dual registration; Medical Device & Biocide
- Subtle apple aroma

SDactiv ZERO has been rigorously tested and proven to be effective against all Bacteria and Yeast. EN 14476 Virucidal testing proves efficacy against all enveloped viruses including Coronavirus, HIV, Hepatitis and Influenza in just 60 seconds.

The revised formula takes advantage of modern, safer and greener chemical components. The new formulation contains a synergistic blend of two powerful biocides, is free from PHMB and contains detergent derived from renewable resources. Creating a simple and effective product for the end user to utilise in the ongoing daily fight against infection.

Bossklein SDactiv ZERO is a 2 in 1 product. Containing a naturally sourced detergent to help clean and remove debris from a surface while simultaneously disinfecting.

The subtle apple aroma leaves surfaces and rooms reassuringly fresh, clean and ready for the next visitor.

COMPOSITION

Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecyldimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes.

INSTRUCTIONS FOR USE

Spray clean surfaces and objects to be disinfected from a distance of 30cm so that they are completely moistened and let the disinfectant take action over 60 seconds (to achieve full efficacy leave for 5 minutes). Allow surface spray to dry or wipe with clean tissue. For professional use only.

REGISTRATION

Dual registered as a Class IIa medical device for use on both non invasive medical devices and as a general purpose Biocide registered disinfectant. This ensures suitability for use within all types of clinical and non clinical environments.

TEST STANDARDS

The following test standards have been achieved: EN 14885, EN 14476, EN 13727, EN13697, EN 13624.

VARIANTS AND PACKAGING

BOS2006500ml Trigger Spray x 10 per boxBOS20051 litre Trigger Spray x 10 per boxBOS20072.5 litre Refill Bottle x 4 per boxBOS20045 litre Refill Bottle x 4 per box

MICROBIOLOGICAL EFFICACY

MICROORGANISM	TIME
Bactericidal	5 min
BS EN 13727:2012 + A2 2015	
BS EN 13697:2015 + A1 2019	
Pseudomonas aeruginosa	5 min
Staphyloccocus aureus	5 min
Enterococcus hirae	5 min
Yeasticidal	5 min
BS EN 13624:2013	
Candida albicans	5 min
Enveloped viruses	60 sec
BS EN 14476:2013 + A1:2013 + A2:2019	
Vaccinia virus	60 sec
Hepatitis B virus (HBV)	60 sec
Hepatitis C virus (HCV)	60 sec
Hepatitis Delta virus	60 sec
HIV	60 sec
Influenza virus	60 sec
Coronavirus	60 sec
Herpesviridae	60 sec
Filoviridae (Ebola)	60 sec
Rubella virus	60 sec
Rabies virus	60 sec
Measles virus	60 sec
Poxviridae	60 sec





Complete **hygiene** solutions

DirectaDentalGroup

* Always test an inconspicuous area first. Do not use on copper or aluminium.

Topdental (Products) Ltd 12 Ryefield Way Silsden West Yorkshire BD20 DEF UK Tel +44 (0)1535 652 750 www.topdental.co.uk v1 0 0822