



ULTRA
160 PREMIUM WIPES
175 x 155mm

V-WIPE ZERO

Surface Disinfectant Wipe

ALCOHOL FREE

EN14476
VIRUS TESTED

100%
ALCOHOL
FREE


CONTACT
60 SEC


APPLE
AROMA

40gsm
THICKER
WIPE

160
PREMIUM
WIPES

KEY FEATURES

Bossklein V-WIPE ZERO wipes are a highly effective cleaner and disinfectant. Designed primarily for the dental and medical sector. Ideal to use where time is critical, such as in between patients and also in high traffic areas.

The brand new alcohol free ZERO formulation is ideal to use on the majority of non porous, wipeable surfaces; even those that are sensitive to alcohol, including dental chairs*

- Contact time of only 60 seconds
- 2 in 1; Cleans and disinfects
- Safe to use on sensitive surfaces*
- Modern, safe formulation
- Dual registration; Medical Device & Biocide
- Subtle apple aroma

ZERO has been rigorously tested and proven to be effective against all Bacteria and Yeast. EN 14476 Virucidal testing proves efficacy against all enveloped viruses including Coronavirus, HIV, Hepatitis and Influenza in just 60 seconds.

The revised formula takes advantage of modern, safer and greener chemical components. The new formulation contains a synergistic blend of two powerful biocides, is free from PHMB and contains detergent derived from renewable resources.

Bossklein V-WIPE ZERO is a 2 in 1 product. Containing a naturally sourced detergent to help clean and remove debris from a surface while simultaneously disinfecting.

The subtle apple aroma leaves surfaces and rooms reassuringly fresh, clean and ready for the next visitor.

COMPOSITION

Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecylidimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes.

INSTRUCTIONS FOR USE

Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action over 60 seconds. Allow surface moisture to dry or wipe with clean tissue. For professional use only.

REGISTRATION

Dual registered as a Class IIa medical device for use on both non invasive medical devices and as a general purpose Biocide registered disinfectant. This ensures suitability for use within all types of clinical and non clinical environments.

TEST STANDARDS

The following test standards have been achieved: EN 16615, EN 14885, EN 14476, EN 13727, EN 13624.

VARIANTS AND PACKAGING

- BOS3000 Tub of 200 Ultra Wipes x 12 per box
- BOS3001 Refill of 200 Ultra Wipes x 12 per box
- BOS3002 Kit of Ultra Wipes - 6 x Refill + 1 x Tub
- BOS3003 Empty Tub for Ultra Wipes x 12 per box

*Always test an inconspicuous area first. Do not use on copper or aluminium.

MICROBIOLOGICAL EFFICACY

MICROORGANISM	CONTACT TIME
Bactericidal	60 sec
BS EN 16615:2015	
BS EN 13727:2012 + A2:2015	
Pseudomonas aeruginosa	60 sec
Staphylococcus aureus	60 sec
Enterococcus hirae	60 sec
Yeasticidal	60 sec
BS EN 16615:2015	
BS EN 13624:2013	
Candida albicans	60 sec
Enveloped viruses	60 sec
BS EN 14476:2013 + A2:2019	
Vaccinia virus	60 sec
Hepatitis B virus (HBV)	60 sec
Hepatitis C virus (HCV)	60 sec
Hepatitis Delta virus (HDV)	60 sec
HIV	60 sec
Influenza virus	60 sec
Coronavirus	60 sec
Herpesviridae	60 sec
Filoviridae (Ebola)	60 sec
Rubella virus	60 sec
Rabies virus	60 sec
Measles virus	60 sec
Poxviridae	60 sec

Wipes are available in our "Ultra" wipe 40gsm premium substrate, for ultimate strength and grip. Each wipe measures 175x155mm and each pack contains 160 Ultra wipes. Both re usable dispenser tubs and economical refill packs are available.



Complete hygiene solutions

DirectaDentalGroup
DIRECTA topdental ORSING parkell physics
RÖNVIG TrollDental Sendoline CONTACTEZ

Topdental (Products) Ltd
12 Ryefield Way Silsden West Yorkshire BD20 0EF UK
Tel +44 (0)1535 652 750 www.topdental.co.uk