


TOUAREG™ - S **Ø6.0**

AB/AE Surface
Titanium Grade 5

Catalog No.	Length mm
ISPS0660	6.25
ISPS0860	8
ISPS1060	10
ISPS1160	11.5
ISPS1360	13

Drill Sequence

BONE TYPE D-IV	BONE TYPE D-II & III	BONE TYPE D-I
1. Tri-Step*	1. Tri-Step*	1. Tri-Step*
2. 3.6	2. 3.6	2. 3.6
3. 4.2	3. 4.2	3. 4.2
4. (5.2)	4. 5.2	4. 5.2
	5. (5.6)	5. 5.6

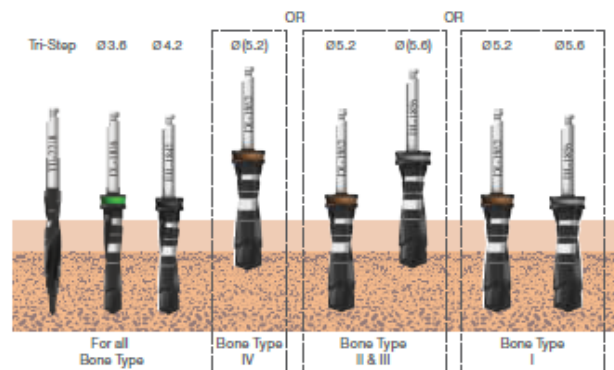

Note: (x.xx) drill to the depth of the cortex only.
All measurements are in mm.

⚠ CAUTION: The drill preparation is up to 1mm longer than the implants.

* For initial drilling, you may use 2, 2.8 and 3.2 drills in sequence instead of the Tri-Step drill.