
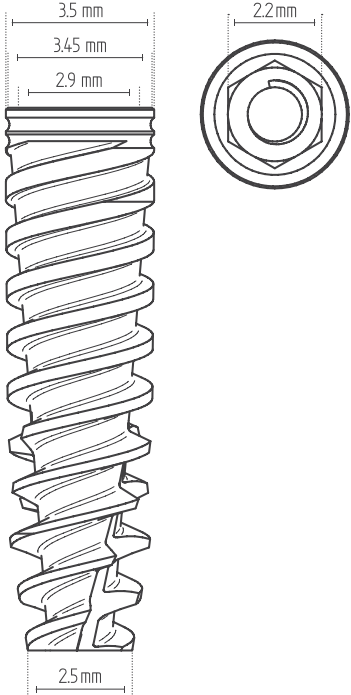



**TOUAREG CloseFit™ - RP**    **Ø3.5**

OsseoFix™ Surface  
Titanium Grade 5

Catalog No.	Length mm		
ISPX0835	8		
ISPX1035	10		
ISPX1135	11.5		
ISPX1335	13		
ISPX1535	15		
ISPX1835	18		

**Drill Sequence**

BONE TYPE D-IV	BONE TYPE D-II & III	BONE TYPE D-I
1. Tri-Step*	1. Tri-Step* 2. (3.2)	1. Tri-Step* 2. (3.2)

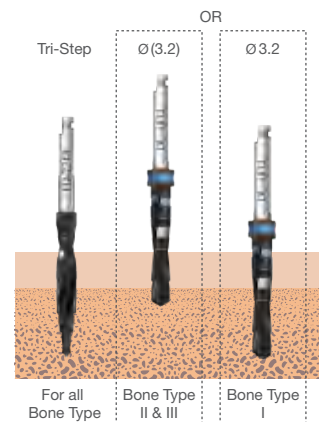

Note: (x.xx) drill to the depth of the cortex only. All measurements are in mm.

⚠ CAUTION: The drill preparation is up to 1mm longer than the implants.

\* For initial drilling, you may use 2, 2.8 and 3.2 drills in sequence instead of the Tri-Step drill.