

## TOUAREG CloseFit™ - WP $\varnothing 4.3$

OsseoFix™ Surface  
Titanium Grade 5

Catalog No.	Length mm		
ISPX0843	8		
ISPX1043	10		
ISPX1143	11.5		
ISPX1343	13		
ISPX1543	15		
ISPX1843	18		

### Drill Sequence

BONE TYPE D-IV	BONE TYPE D-II & III	BONE TYPE D-I
1. Tri-Step* 2. (3.6)	1. Tri-Step* 2. (3.6)	1. Tri-Step* 2. 3.6

Note: (x.xx) drill to the depth of the cortex only. All measurements are in mm.

⚠ CAUTION: The drill preparation is up to 1mm longer than the implants.

\* For initial drilling, you may use 2, 2.8 and 3.2 drills in sequence instead of the Tri-Step drill.

