
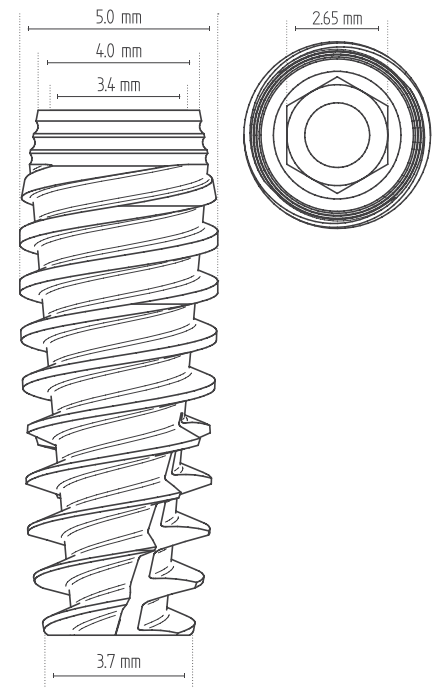


TOUAREG CloseFit™ - WP **Ø 5.0**

OsseoFix™ Surface
Titanium Grade 5

Catalog No.	Length mm		
ISPX0849	8		
ISPX1049	10		
ISPX1149	11.5		
ISPX1349	13		
ISPX1549	15		
ISPX1849	18		

Drill Sequence

BONE TYPE D-IV	BONE TYPE D-II & III	BONE TYPE D-I
1. Tri-Step*	1. Tri-Step*	1. Tri-Step*
2. 3.6	2. 3.6	2. 3.6
3. (4.2)	3. 4.2	3. 4.2
	4. (4.6)	4. 4.6

Note: (x.xx) drill to the depth of the cortex only.
All measurements are in mm.

⚠ CAUTION: The drill preparation is up to 1mm longer than the implants.

* For initial drilling, you may use 2, 2.8 and 3.2 drills in sequence instead of the Tri-Step drill.

