

PERASAFE® INSTRUMENT STERILENT POWDER

USER GUIDE

A safer form of peracetic acid which quickly and effectively decontaminates selected medical devices. An Aldehyde-free formulation that does not harm equipment or human health and does not require special fume extraction or waste disposal systems for use.

The sequences shown here should be used as guidance. Please refer to local hospital guidelines for further information.



01 Fill container with lukewarm tap water (30-35°C) to the required volume.



02 Add powder to water direct from container or required amount using measuring cap provided.



03 Stir until the powder dissolves to form a clear blue solution (10-15 minutes).



04 Pour the solution into an immersion bath or endoscope reprocessor for use. The liquid will fade to a colourless solution in approximately one hour but can be used for up to 24 hours after activation, i.e. when the clear blue solution is achieved.

