


HU-FRIEDY
PROGRAMS

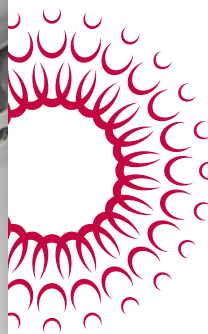

SWISS PERIO KIT

Developed in collaboration with
Swiss Perio Group, Dr. Rino Burkhardt

How the best perform





HU-FRIEDY KEY OPINION LEADERS

DR. RINO BURKHARDT

swiss+perio

SwissPerio Education was founded by a group of periodontists and stands for a new training platform offering tailored courses on various topics in the domain of periodontology and related fields. Events organized by SwissPerio are different from those of other providers as the transfer of disciplinary knowledge on an expert and evidence-based level is just one of the facets of the courses. The restricted number of participants provides room for personal interaction and open discussions, stimulating the trainees for critical thinking and teaching them in clinical decision-making.

Most of the SwissPerio events are related to clinical practice and, therefore, the third focus of SwissPerio Education is directed at the improvement of psychomotor manual skills.

SWISSPERIO EDUCATION FOR RESPONSIBLE CLINICAL ACTION

Rino Burkhardt, one of the Board Members of SwissPerio Education, is responsible for the training programs. He is an EFP (European

Federation of Periodontology) certified specialist in periodontology and received his Masters degree from the Medical Faculty of the University of Berne, Switzerland.



Since 1996 he runs a private clinic in Zurich, limited to periodontology and implantology. Additionally, he holds an Honorary Professorship at the University of Hong Kong and a lectureship at the University of Zurich. He is an active member of the Swiss Society of Periodontology (SSP), the European Academy of Aesthetic Dentistry (EAED), the European Association for Osseointegration (EAO), and Board Member of the Swiss Society of Oral Implantology (SGI). He published numerous book chapters and articles and won a research prize from the EFP.

swissperio.com



SWISS PERIO KIT

FEATURES AND BENEFITS

SPECIFIC DESIGNED INSTRUMENTS

Less tissue trauma by refined working ends;
Ultimate manufacturing tolerances
for precise handling;
Easy to clean.

BLACK INSTRUMENTS / BLACK LINE EXTENSION

More contrast between the
instrument and the tissue;
Avoid light reflection;
More sharpened than standard.

BALANCED WEIGHT

More control and precision
in specific and delicate
microsurgery procedures.






SWISS PERIO KIT PERIOSTEAL ELEVATORS



- ① **SPP24GSPX**
- ② **SPP24GSICX**
- ③ **SPP24GSP2X**

Three double-ended periosteal elevators are equipped with different working ends based on the variability of oral mucosa, and are designed to cover all kinds of flap elevations from delicate, tiny papilla to coarse palatal masticatory mucosa. The sharpness of the elevators and the ultimate flat, however stiff configuration, guarantee a minimally-traumatic flap mobilization. The blunt working ends with a groove are made to retract full thickness flaps and provide a stable seat on bony surfaces and crests.

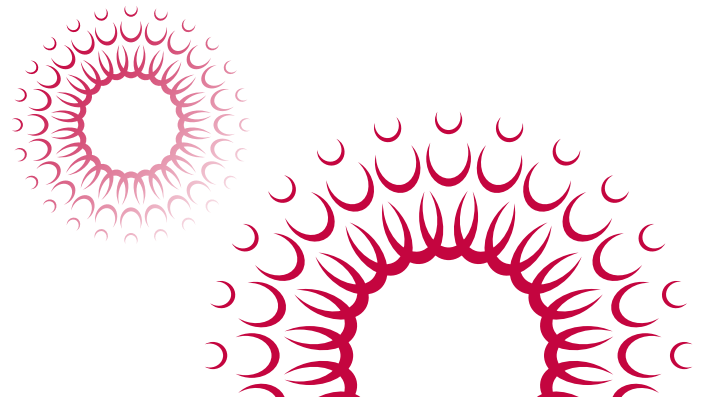



PAPILLA ELEVATOR & TISSUE DISSECTORS





SPPSCHLEEX

The SPPSCHLEEX pushes and pulls the papilla. It is indispensable when the interdental area has to be accessed. After intrasulcular sharp dissection of the papilla, its basis and the gingiva of the col area, the interdental mucosa can be elevated and mobilized under ultimate controlled conditions. A key factor for achieving primary wound closure and maintaining wound stability.



SPPH2X

In some surgical sites, such as former extraction sockets, the mucosa is firmly adhering to the bone and thus difficult to elevate. The tissue dissector SPPH2X allows a controlled dissection of the flap from the underlying hard tissue wound bed. With the opposite end of the instrument, the mucosal tissues can be elevated, directed from posterior to anterior with pulling instead of pushing strokes.

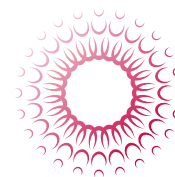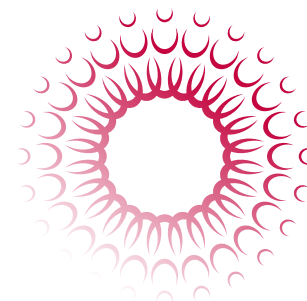



SWISS PERIO KIT MICROSURGICAL PLIERS



SPTPDSSPV

The working ends of the surgical forceps SPTPDSSPV are configured with micro teeth, allowing to firmly grasp the oral mucosa without traumatizing the flap or graft margins unnecessarily.



SPTPDAPV

The anatomical forceps SPTPDAPV are designed for suturing and tightening the knots, but can also be used for manipulating the soft tissue. The very fine and precise configuration of the working ends guarantee a firm seat of the threads and minimize tissue trauma during the surgical procedure.





MICROSURGICAL SCISSORS



SPSPV

The micro tissue scissors, SPSPV with its sharp, slightly curved working end is made for fine resections of soft tissues and trimming of wound margins or mucosal grafts.

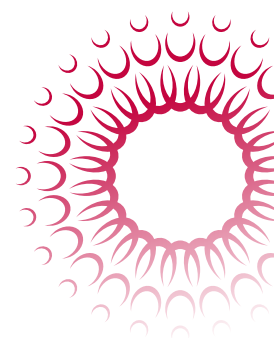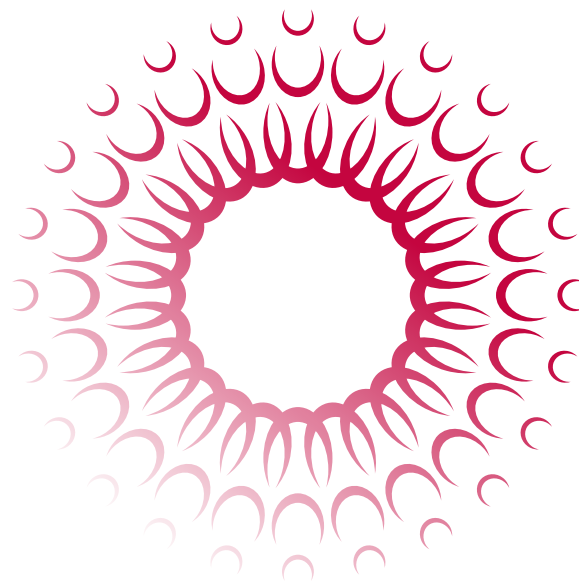





SWISS PERIO KIT MICROSURGICAL SCISSORS



SPSPV45

The suture scissors SPSPV45 is exclusively designed for cutting threads. The inclined working ends allow access to difficult areas even far distal to molars in posterior zones.

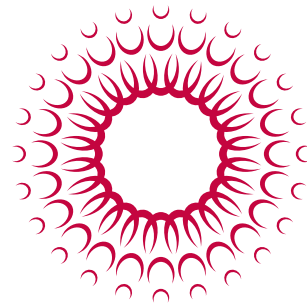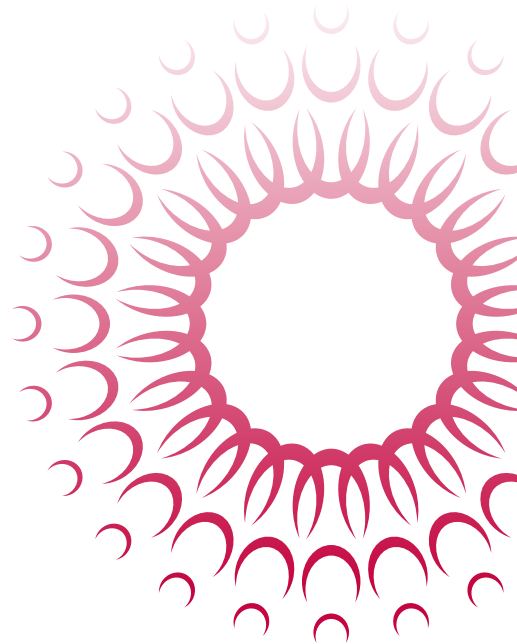




NEEDLE HOLDER



SPNHDPVN

The needle holder SPNHDPVN is completely revised in all details and unmatched in its precision. It allows a firm seat of smaller needles and provides an easy handling of suture diameters in a range of 5-0 to 9-0. The instrument is equipped with a lock that facilitates finger rotation without applying pressure, leading to a more precise execution of finger and hand movements. Additionally, the low locking forces reduce hand tremor while grasping the needles.

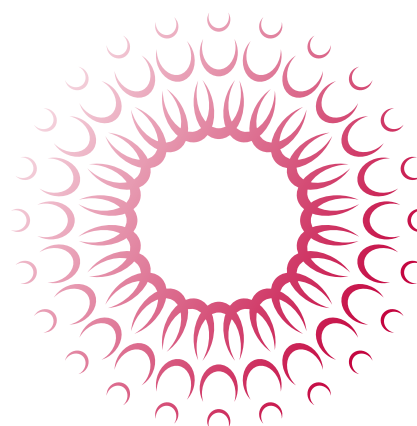



SWISS PERIO KIT THE KIT

HU-FRIEDY IS PROUD TO INTRODUCE YOU THE COMPLETE SWISS PERIO KIT

PRODUCT	CODE
Complete Swiss Perio Kit	SPKITDIN
THE SWISS PERIO KIT INCLUDES:	
IMS Infinity DIN Surgical cassette for 18 instruments, black	IMEDIN187S
Swiss Perio Microsurgical Needle Holder Stainless Steel	SPNHDPVN
Swiss Perio Microsurgical Plier Straight Stainless Steel	SPTPDSSPV
Swiss Perio Microsurgical Plier Straight Stainless Steel	SPTPDAPV
Swiss Perio Microsurgical Scissor Stainless Steel	SPSPV
Swiss Perio Microsurgical Scissors Stainless Steel, 45 degree	SPSPV45
Swiss Perio Periosteal Elevators Mod. P24G, Black Line	SPP24GSPX
Swiss Perio Periosteal Elevators Mod. P24GSIC, Black Line	SPP24GSICX
Swiss Perio Tissue Dissector Mod. PH2, Black Line	SPPH2X
Swiss Perio Papilla Elevator Mod. PPSCHLEE, Black Line	SPPSCHLEEX
84 Lucas Double Ended Surgical Curette, Black Line	CL84X
Swiss Perio Periosteal Elevators Mod. P24GSP, Black Line	SPP24GSP2X
YG7/8 Younger-Good, EverEdge 2.0, Smooth Handle Stainless Steel	SYG7/8CHE2
Colorvue Probe Handle, Smooth Handle Stainless Steel	PH6CH
Mini Scalpel Blade / Mirror Handle Stainless Steel	SHDPV
5 European Style Round Scalple Handle	10-130-05E
N.6 Simple Steam Mirror Handle, Smooth Handle	MHE6CH
17 Dressing Pliers	DP17
Q2N Color-Coded Nabers Probe, Smooth Handle	PQ2NCH
12 Colorvue Probe tips, 12 Replacements Tips	PCV12PT
European Simple Steamn.5 Double-Sided HD Mirror, Single	MIR5DSHDE
KIT IS ALSO AVAILABLE WITH IMS INFINITY CONTAINER CASSETTE:	
Complete Swiss Perio Kit with container cassette (2 x IMECC120)	SPKITCC





INSTRUMENT MANAGEMENT SYSTEM


By standardising your instruments with IMS, additional time and revenue is generated while extending the useful life of your Hu-Friedy instruments.

Dental professionals state that by implementing IMS, they save 5 to 10 minutes per procedure allowing the staff to spend more time on value-added activities.

In addition, safer transportation of soiled instruments reduces risk of injury and exposure to blood borne pathogens.

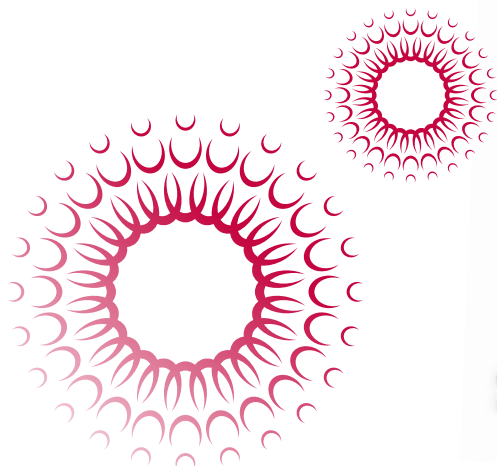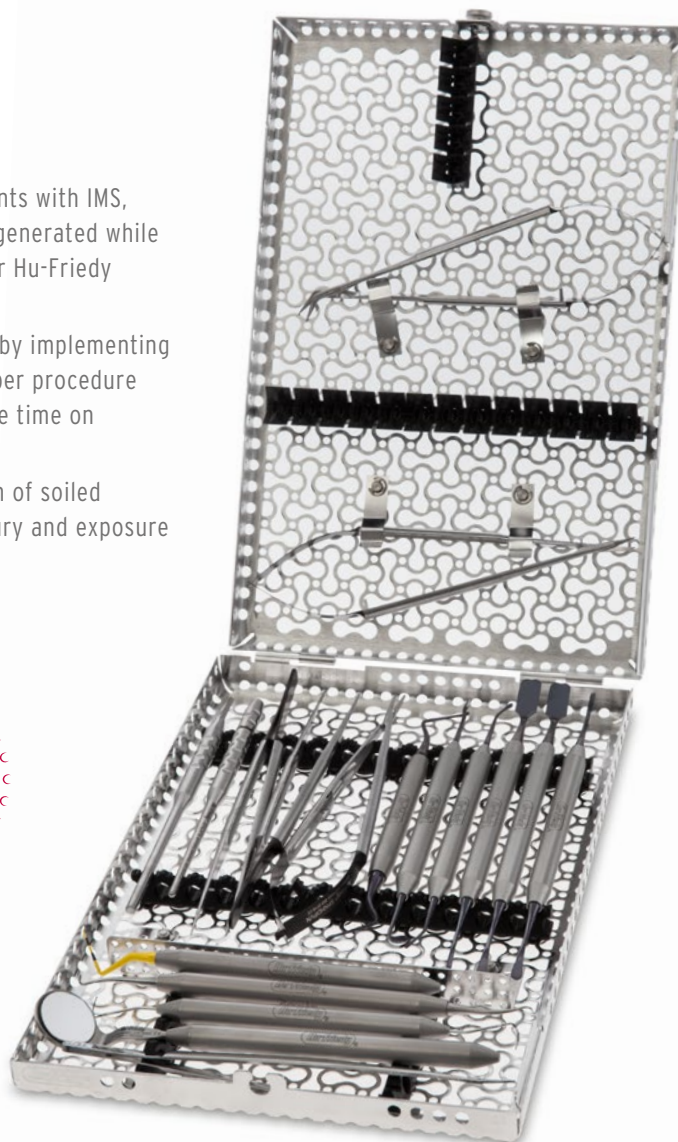

For more information about
Hu-Friedy's Total Solutions visit
HU-FRIEDY.EU

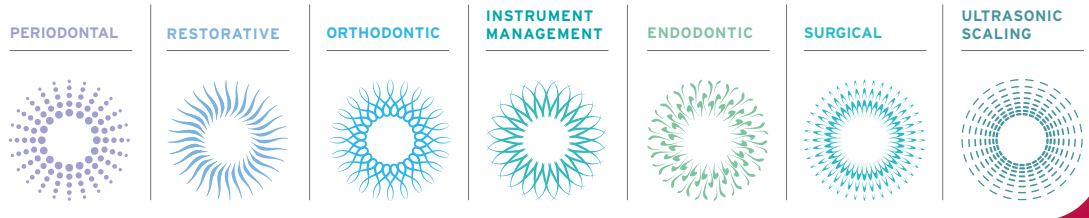

Manufacturer: Hu-Friedy Mfg. Co., LLC | 3232 N. Rockwell Street | Chicago, IL 60618 | USA
Customer Care Department: Hu-Friedy Mfg. Co., LLC.
Free Call +800 HUFRIEDY (00800 48 37 43 39) | Free Fax 00800 48 37 43 40
E-Mail: info@hu-friedy.eu | Website: Hu-Friedy.eu

©2018 Hu-Friedy Mfg. Co., LLC. All rights reserved. The contents of this Product Catalog & Reference Guide including, but not limited to, all text, photographs and graphics, are the sole and exclusive property of Hu-Friedy Mfg. Co., LLC and may not be copied, reproduced or distributed without prior written permission. N2310-EN/0218
Hu-Friedy Mfg. Co., LLC. reserves the right to change features of products, images and text at any time without notice.



How the best perform