

Touareg™ - S Ø5.0


AB/AE Surface

Titanium Alloy - Ti 6Al 4V ELI

Catalog No.	Length mm	
ISPS0650	6.25	
ISPS0850	8	
ISPS1050	10	
ISPS1150	11.5	
ISPS1350	13	
ISPS1650	16	

Drill Sequence

BONE TYPE D-IV	BONE TYPE D-II & III	BONE TYPE D-I
1. Tri-Step*	1. Tri-Step*	1. Tri-Step*
2. 3.6	2. 3.6	2. 3.6
3. (4.2)	3. 4.2	3. 4.2
	4. (4.6)	4. 4.6

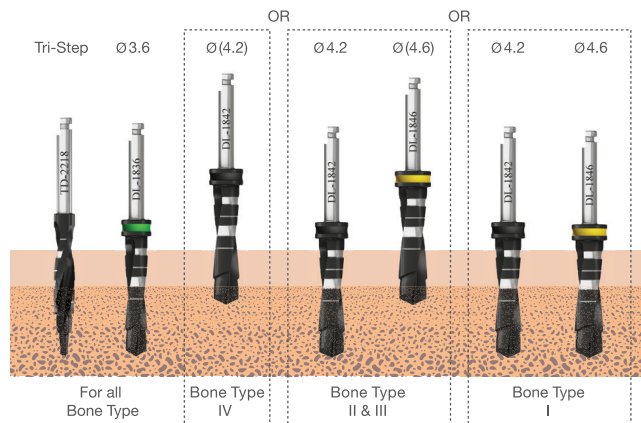

Note: (x.xx) drill to the depth of the cortex only.

All measurements are in mm.

⚠ CAUTION: The drill preparation is up to 1mm longer than the implant.

* For initial drilling, you may use 2, 2.8 and 3.2 drills in sequence instead of the Tri-Step drill.

Radiographic Template