

LUBRIMED MEDICAL GREASE

Medical grease for turbines with ball bearings.
Packed in boxes of 6 cartridges. Turbines are lubricated using the Lubrimed greaser.

IMPORTANT: unsuitable for Eolia turbines.



First-time use:

Remove the cap and rotate the knurled rear section until grease appears at the tip.
Wipe off excess grease and put the cap back on after use.



Lubricate before each sterilisation or at least twice a day.

Remove the bur and insert the tip of the greaser as far as it will go. The dose required corresponds to a half-turn of the knurled rear section.

Use the reference marks.



For more details, please refer to the operating instructions for the instruments and devices.