

Imprint™ 4 VPS Impression Material – Portfolio overview

Product	Dispensing system	Viscosity low  high	Setting version	Working time (23 °C) 	Intra-oral syringing time (37 °C) 	Intra-oral setting time (37 °C) 
TRAY MATERIAL						
Imprint™ 4 Penta™ Putty		Putty	Regular Set	1:30	–	2:30
Imprint™ 4 Penta™ Heavy		Heavy Body	Regular Set	2:00	–	2:00
Imprint™ 4 Penta™ Super Quick Heavy		Heavy Body	Fast Set	1:15	–	1:15
WASH MATERIAL						
Imprint™ 4 Super Quick Ultra-Light			Fast Set	1:15	0:35	1:15
Imprint™ 4 Light			Regular Set	2:00	1:00	2:00
Imprint™ 4 Super Quick Light			Fast Set	1:15	0:35	1:15
Imprint™ 4 Regular			Regular Set	2:00	1:00	2:00
Imprint™ 4 Super Quick Regular			Fast Set	1:15	0:35	1:15

Imprint™ 4 VPS Impression Material – Material combinations per technique

Impression technique	Tray material	Recommended wash material	Wash material
1-step technique			
Imprint™ 4 Penta™ Heavy hydrophilic heavy body		Imprint™ 4 Regular	
		Imprint™ 4 Light	
Imprint™ 4 Penta™ Super Quick Heavy fast setting hydrophilic heavy body		Imprint™ 4 Super Quick Regular	
		Imprint™ 4 Super Quick Light	
Imprint™ 4 Penta™ Putty putty consistency		Imprint™ 4 Regular	
2-step technique			
Imprint™ 4 Penta™ Putty putty consistency		Imprint™ 4 Super Quick Ultra-Light	
		Imprint™ 4 Light	
Imprint™ 4 Penta™ Super Quick Heavy fast setting hydrophilic heavy body		Imprint™ 4 Super Quick Light	
		Imprint™ 4 Light	