
Pentamix[™] Lite

Automatic Mixing Unit

Fast and intuitive set-up and use

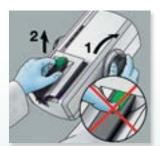

1. Remove the cartridge.




2. Insert Penta™ Foil Bag with the 3M ESPE impression material of your choice into the cartridge.



 Insert the cartridge into the Pentamix™ Lite Automatic Mixing Unit.




4. Turn the wheel (piston) forward.

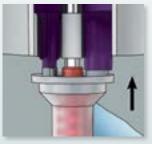

5. Attach a Penta™ Mixing Tip - Red.



6. Close the latch with an audible click.

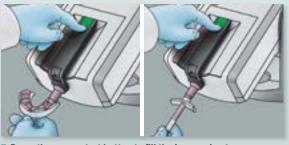

7. Press the green start button to fill the impression tray or elastomer syringe.



Pentamix™ Lite Automatic Mixing Unit

For optimum performance, 3M ESPE only recommends the use of 3M ESPE Penta™ Impression Materials in the Pentamix™ Lite Automatic Mixing Unit. Non original materials can damage the device!

3M ESPE Impression Material		Max. working time from start of mixing (min:sec)	Setting time in mouth (min:sec)	Setting time from start of mixing (min:sec)	
POLYETHER IMPRESSION MATERIALS					
6	Impregum™ Penta™	(3)	1:45*	_	= 6:00
6	Impregum™ Penta™ Soft		1:45*	_	= 6:00
VPS IMPRESSION MATERIALS					
6	Imprint™ 4 Penta™ Putty	(3)	1:30	2:30	_
6	Imprint™ 4 Heavy		2:00	2:00	_
6	Imprint™ 4 Penta™ Super Quick Heavy	(A)	1:15	1:15	_

^{*} Clinically recommended working time.

Penta™ Mixing Tip – Red for all Pentamix™ Devices

Always use 3M ESPE Penta Mixing Tips — Red in your Pentamix device to ensure proper mixing and impression results.





Dental Products

2510 Conway Avenue St. Paul, MN 55144-1000 USA 1-800-634-2249

3M Canada

Post Office Box 5757 London, Ontario N6A 4T1 Canada 1-888-363-3685 3M, ESPE, Impregum, Imprint, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2014. All rights reserved. Please recycle. Printed in U.S.A. 70-2013-0606-8