



Fit Checker™ Advanced & Fit Checker™ Advanced Blue



The simplest
way towards
successful
finalisation of your
prosthetic work

For a simple, easy and
precise check of the fit
of prosthetic work.

GC



Fit Checker™ Advanced Blue

The ideal blue colour gradation for

- Fit checking of all kinds of prosthetic works, especially resin, ceramic and zirconia-based crowns and bridges.
- Bite registration and evaluation of occlusal contacts.



Balanced thixotropy allows for a **non-runny consistency**. Under pressure, Fit Checker Advanced Blue offers **easy flow** and **extremely thin film thickness** for a perfect fit.



Misfits are easily detected thanks to **optimal transparency** and unique **blue colour gradation**.



Finally, it's possible to make certain of a **perfect fit for indirect restorations**, avoiding marginal gaps and major occlusal adjustments after cementation.

1


Snap set

Sufficient working time and rapid setting (1 min. in the mouth) for a fast and comfortable procedure.

2

Clear and precise visualisation

A perfect colour gradation for immediate visualisation of occlusal contacts and poorly fitting areas.

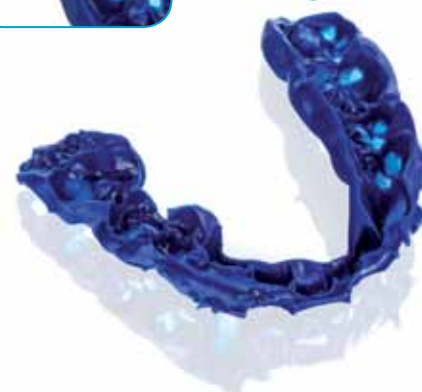



Fit Checker™ Advanced

The ideal white colour gradation for

- Fit checking of all kinds of prosthetic works, especially complete and partial dentures and metal-based crowns and bridges.



The perfect solution for checking the adaptation of complete and partial dentures.



Fit Checker Advanced offers ideal consistency and high wettability for **easy flow** into denture bases and oral tissues. Fit Checker Advanced is easily **removed in one step**.



Denture misfits can then be corrected with acrylic reliners. After relining, final checking is performed to assure **patient comfort**.

3

No mess during adjustments

The set material does not get caught by the bur while relieving pressure spots.

4

Easy to peel off

The silicone pellicle peels off easily for a clean and fast procedure.



The simplest way towards successful finalisation of your prosthetic work

Fit Checker Advanced and Fit Checker Advanced Blue from GC provide a simple and precise way to assess the fit of prosthetics and make timely adjustments at the fitting/seating appointment - avoiding potential future costs and ensuring patient comfort.

Fit Checker™ Advanced	Benefit	Fit Checker™ Advanced Blue
VPES Silicone	Innovative silicone material with intrinsic wettability, flowability and resistance	VPES Silicone
White	Two different colours to meet all clinical requirements	Blue
Working time: 1 min. Setting time: 1 min. in mouth	Sufficient working time and a short setting time in the mouth	Working time: 1 min. Setting time: 1 min. in mouth
Handmix (tubes) or Automix (cartridges)	Automix cartridges available for an easy, clean and fast procedure	Automix cartridges



Packages

GC Fit Checker™ Advanced / GC Fit Checker™ Advanced Blue	
004903	Fit Checker Advanced, 1-1 Pack (55 g + 50 g), White + retarder (3g) + accessories
004904	Fit Checker Advanced, Cartridges (2 x 62 g), White + mixing tips IIS (3) and IISS (3)
004905	Fit Checker Advanced Blue, Cartridges (2 x 56 g), Blue + mixing tips IIS (3) and IISS (3)
800006	GC Cartridge Dispenser II
890130	GC Mixing Tips II, 60 x SS (Yellow)
800100	GC Mixing Tips II, 60 x S (Pink)

GC EUROPE N.V.

Head Office
Researchpark
Haasrode-Leuven 1240
Interleuvenlaan 33
B-3001 Leuven
Tel. +32.16.74.10.00
Fax. +32.16.40.48.32
info@gceurope.com
http://www.gceurope.com

GC UNITED KINGDOM Ltd.

12-15, Coopers Court
Newport Pagnell
UK-Bucks. MK16 8JS
Tel. +44.1908.218.999
Fax. +44.1908.218.900
info@uk.gceurope.com
http://uk.gceurope.com

