


Sendoline[®]

S3 System

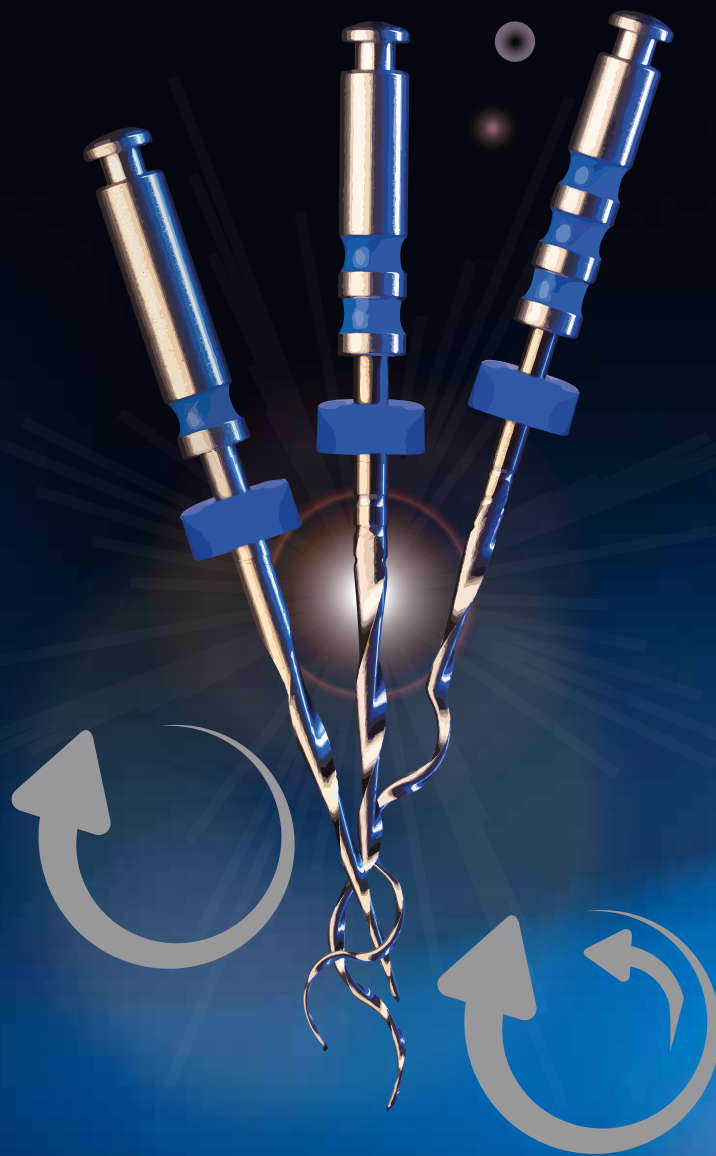

ONE SYSTEM — TWO MOVEMENTS

Continuous rotation

Reciprocating movement

STEP-BY-STEP GUIDE



1. Working length (WL)

Use an ISO #10 manual hand file to establish working length.

2. S3 #1 Coronal flaring

Will only work in the coronal, first 1/3 of the root canal.

3. S3 Glidepath file

- will work until WL is reached
Establish a glide path respecting the original root canal anatomy, as well as ensuring a safe preparation of the root canal.

4. S3 #2 & #3

Finalize the root canal preparation using crown down technique. In many cases you will reach WL already with #2. In more severe root canal curvature, you need to continue with #3.



GP 20/04 #1 30/08 #2 30/06 #3 30/04

FULL FLEXIBILITY

Each of Sendoline's S3 instruments are manufactured in NiTi alloy with different flexibility adapted for the role it plays during the root canal treatment. This is unique for the Sendoline S3 System.

The first instrument in the sequence, Sendoline S3 #1 with its traditional NiTi alloy, ensures good straight-line access. Instruments #2 and #3, including the additional S3 Glidepath are heat-treated. The heat-treated NiTi alloy together with the dimension of the instrument is giving each instrument in the sequence the perfect balance between flexibility and efficiency.

Sendoline S3 System with Sendoline Endo Motor

The Sendoline S3 System can be used with most endodontic motors. The Sendoline Endo Motor have all settings for the Sendoline S3 System pre-installed.

If you prefer to only work with reciprocating movement the most convenient is to use the Sendoline S1 contra-angle handpiece which you easily connect directly to your dental unit.



All S3 instruments are designed with the unique S-cross section by Sendoline.

PERFECT ENDO

PRODUCT OVERVIEW

REF	Product description	Pieces
S3193008BS	Sendoline S3 System #1 30/08 - 19 mm	6 pcs/box
S3253006BS	Sendoline S3 System #2 30/06 - 25 mm	6 pcs/box
S3253004BS	Sendoline S3 System #3 30/04 - 25 mm	6 pcs/box
S3313006BS	Sendoline S3 System #2 30/06 - 31 mm	6 pcs/box
S3313004BS	Sendoline S3 System #3 30/04 - 31 mm	6 pcs/box
S3252004BS	Sendoline S3 Glidepath 20/04 - 25 mm	6 pcs/box
S3312004BS	Sendoline S3 Glidepath 20/04 - 31 mm	6 pcs/box
S301BS	Sendoline S3 System Assortment*	4 pcs/box
S3S2222	Sendoline Endo Motor S3 Start Up Set**	Set
222200	Sendoline Endo Motor	Kit
6542472	S1-E Contra-angle handpiece S3 Assortment***	Kit

*Content: 1x S3 #1 30/08 19mm, 1x S3 Glidepath 20/04 25mm, 1x S3 #2 30/06 25mm, 1x S3 #3 30/04 25mm

**Content: 1x Sendoline Endo Motor, 1x S3 #1 30/08 19mm, 1x S3 #2 30/06 25mm, 1x S3 #3 30/04 25mm, 1x S3 Glidepath 20/04 25mm

***Content: 1x S1-E Contra-angle handpiece, 1x S1 Apex clip 2.0, 1x Sendoline S3 System Assortment



Scan the QR code for more information about Sendoline S3 System www.sendoline.com

The contents of this brochure including layout and images are protected by copyright. ©2023 Directa AB. All rights reserved.

BRO_SendolineS3_en_2023-02-20_REV01

DIRECTA