

One.
Not two
or six.

wave•one™

For better dentistry

DENTSPLY
MAILLEFER

Control. ●

Don't you think life is better when you are in control?

The **WaveOne™** is a new endodontic system that uses reciprocating technology. This means that it will only shape the canal when pressure is applied, giving you that feeling of complete control.

WaveOne™ also simplifies the procedure, requiring just one premium quality NiTi instrument per root canal procedure in most cases. The perfect solution to those wishing to switch from hand files to a rotary based system.

WaveOne™ from **DENTSPLY**...of course.

Why choose **WaveOne™**

- Only one NiTi instrument required per root canal procedure in most cases
- Triangular cross section and reciprocation
- M-Wire

The combination of these features give you that feeling of complete control over costs, time taken, clinical outcomes, the comfort of your patients and the management of your inventory.




Strong.

When endodontics becomes synonymous with strength.

The geometry of the **WaveOne™** files and the **WaveOne™** proprietary DENTSPLY M-Wire™ Nickel-Titanium technology are the main reasons for the superior performance* of the **WaveOne™** file compared to standard NiTi files.

Fatigue resistance*
time until breakage (s)

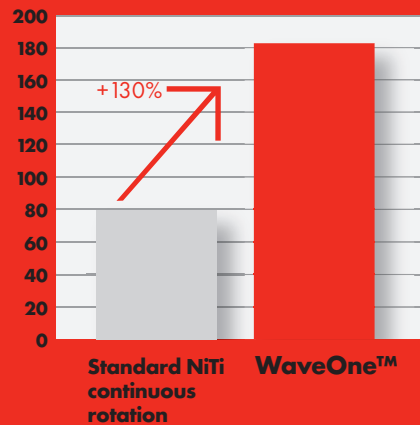

DENTSPLY utilises an innovative thermal treatment process to create the M-Wire™ NiTi material. Its resulting characteristics provide overall flexibility and greater resistance to cyclic fatigue, the leading cause of file separation. In fact, the tougher the instrumenting challenge, like shaping severely curved to the apex, the more apparent the M-Wire™ NiTi's advantages become.

* Data on file.

Simplified.

What Dr. Julian Webber says;



Dr **JULIAN WEBBER**
BDS MSc DGDGP FICD

Specialist in Endodontics

WaveOne™ **SINGLE FILE** **RECIPROCATING** **SYSTEM**

“**WaveOne™** will appeal to all colleagues, especially GDPs who are looking for a simplified method of Root canal preparation, with the well known advantages of NiTi endo files. Perfect canal shapes are achieved with just 1 single file... what more would you want?”



Supported.

Case Study

Conducted by **Dr. Julian Webber**

BDS MSc DGDP FICD

The tooth presents symptoms of irreversible pulpitis and early apical periodontitis.

Initial assessment is narrow and curved canals radiographically. The tooth has 4 canals.

Access is made.

All canals worked to length with 10 K file.

WaveOne™ Primary (25 .08) is chosen.

Length is reconfirmed with 10K File.

WaveOne™ Primary to length in all 4 canals.

Obturation with Warm Vertical Condensation using Calamus Dual.

International OPLs

- **Pierre Machtou**
- **Sergio Kuttler**
- **Willy Pertot**
- **Julian Webber**
- **Cliff Ruddle**
- **Ben Johnson**
- **John West**



Usage.

Directions for use

01

If a size 10 is very resistant to movement use **WaveOne™**
Small 21:06

02

If a size 10 easily moves to length use **WaveOne™**
Primary 25:08

03

If a size 20 hand file or larger goes to length use **WaveOne™**
Large 40:08

Guidelines for use

1. Use **WaveOne™** files with a progressive up and down movement no more than 3-4 times. Little force is required.
2. Remove file regularly, wipe clean, irrigate and continue.
3. If file does not progress confirm patent canal and consider going to a smaller **WaveOne™** file.
4. In severely curved canals complete apical preparation by hand if reproducible glide path is not possible.
5. Never work in a dry canal and constantly irrigate with NaOCl and later EDTA
6. As preparation time is short activate the irrigating solutions to enhance their effect.



Codes.

Description	Item Code	Pack Size	DSRP £
WaveOne™ Small reciprocating file 21mm	A070022100S01	3 files	£32.70
WaveOne™ Small reciprocating file 25mm	A070022500S01	3 files	£32.70
WaveOne™ Small reciprocating file 31mm	A070023100S01	3 files	£32.70
WaveOne™ Primary reciprocating file 21mm	A070022100P01	3 files	£32.70
WaveOne™ Primary reciprocating file 25mm	A070022500P01	3 files	£32.70
WaveOne™ Primary reciprocating file 31mm	A070023100P01	3 files	£32.70
WaveOne™ Large reciprocating file 21mm	A070022100L01	3 files	£32.70
WaveOne™ Large reciprocating file 25mm	A070022500L01	3 files	£32.70
WaveOne™ Large reciprocating file 31mm	A070023100L01	3 files	£32.70
WaveOne™ Assorted reciprocating file 21mm	A070022190A01	3 files	£32.70
WaveOne™ Assorted reciprocating file 25mm	A070022590A01	3 files	£32.70
WaveOne™ Assorted reciprocating file 31mm	A070023190A01	3 files	£32.70
WaveOne™ Sterile Paper Points Small	A024W00000S01	180	£16.13
WaveOne™ Sterile Paper Points Primary	A024W00000P01	180	£16.13
WaveOne™ Sterile Paper Points Large	A024W00000L01	180	£16.13
WaveOne™ Sterile Paper Points Assorted	A024W00090001	180	£16.13
WaveOne™ Gutta Percha Points Small	A024X00000S00	60	£10.37
WaveOne™ Gutta Percha Points Primary	A024X00000P00	60	£10.37
WaveOne™ Gutta Percha Points Large.	A024X00000L00	60	£10.37
WaveOne™ Gutta Percha Points Assorted	A024X00090000	60	£10.37
WaveOne™ Thermafil Small 6 pack	A170000010S00	6	£21.04
WaveOne™ Thermafil Primary 6 pack	A170000010P00	6	£21.04
WaveOne™ Thermafil Large 6 pack	A170000010L00	6	£21.04
WaveOne™ Thermafil Small 20 pack	A170000011S00	20	£68.63
WaveOne™ Thermafil Primary 20 pack	A170000011P00	20	£68.63
WaveOne™ Thermafil Large 20 pack	A170000011L00	20	£68.63

DSRP - DENTSPLY suggested retail price

Description	Item Code	Pack Size	DSRP £
WaveOne™ Starter Kit 1	A102110000000	1 motor kit	£1,113.03
WaveOne™ Starter Kit 2	A102120000000	1 motor kit	£1,168.14
WaveOne™ Contra-angle 6:1	A101900000000	1	£432.08
WaveOne™ Battery Charger	A102000000100	1	£46.00
WaveOne™ WaveOne Foot Pedal	A102000000200	1	£35.46

WaveOne™ Endo Motor Kit 1

WaveOne™ Reciprocating Files

- 3 blisters PRIMARY 25mm (9 files)
- 1 blister SMALL 25mm (3 files)
- 1 blister LARGE 25mm (3 files)

WaveOne™ GPP 1x Asst'd

WaveOne™ Paper Points 1x Asst'd

Training blocks x 8

Clinical Procedure Card

WaveOne™ DVD

WaveOne™ Endo Motor Kit 2

WaveOne™ Reciprocating Files

- 3 blisters PRIMARY 25mm (9 files)
- 1 blister SMALL 25mm (3 files)
- 1 blister LARGE 25mm (3 files)

WaveOne™ GPP 1x Asst'd

WaveOne™ Paper Points 1x Asst'd

Training blocks x 8

Clinical Procedure Card

WaveOne™ DVD

WaveOne™ Thermafil Obturators

- 1 box SMALL (6x)
- 1 box PRIMARY (6x)
- 1 box LARGE (6x)

WaveOne™ from DENTSPLY...**of course.**

To find out more, visit dentsply.co.uk/waveone
or call **+44 (0)800 072 3313**



For a limited period only, order your **WaveOne™** kit
at dentsplyrewards.co.uk and receive £150 in rewards

To view forthcoming **WaveOne™** courses, please visit
dentsplyacademy.co.uk/upcoming.asp

wave•one™

For better dentistry

DENTSPLY
MAILLEFER